EN Intended Use

The Custom Breast Flange when used in place of the standard size breast flange (included with the **Ameda** HygieniKit Milk Collection System), provides a larger nipple tunnel for added comfort and effectiveness for women who need a larger flange for pumping.

When used with an **Ameda** manual or electrically powered breast pump, it is intended to express and collect milk from the mother's breast to alleviate engorgement of the breast, maintain the ability of lactation, and provide mother's milk for future feedings when separation of mother and baby occurs.

Please inspect the components for cracks or chips as they may affect the functioning of the product.

If discovered, do not use. Contact your local Ameda Retailer for further guidance.

Instructions for Use


Before using the first time, clean product by hand washing in hot soapy water followed by a hot water rinse or in the top rack of a standard household dishwasher. *Do not use solvents or abrasives.* Repeat same steps after each use.

To sanitize before first use, boil the product for 20 minutes; then immediately remove from water to air dry and cool.

Checking Flange Fit


Some mothers do well with the standard size (25 mm) breast flange. You may need a larger flange (30.5 mm) or insert (28.5 mm) if pumping hurts, even on low suction. If the standard flange is too tight, a larger flange or insert may feel better and pump more milk.

To check your flange fit, watch your nipple during pumping.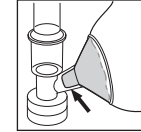

Good Fit

If you see space around your nipple, you have a good fit.

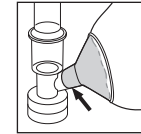Too Tight

If your nipple rubs against the sides of the flange, it is too tight.

To use the larger flanges in your kit:

- Replace standard flange with larger flange.
- Move valve to larger flange.
- See instructions to assemble other parts.
- Try the larger flange with and without the insert on each breast until you find the size that feels better and/or pumps more milk.

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Distributed in USA by:
Evenflo Company, Inc.
1801 Commerce Drive
Piqua, Ohio 45356
1.866.99.AMEDA (26332)

Distributed in Canada by:
Distribue' au Canada par :
Mothers Choice Products Ltd.
2133-11871 Horseshoe Way
Richmond, BC V7A5H5
1.800.604.6225

Distributed in Australia by:
Advantage Health Care
34 Percy Street
Mordialloc, VIC 3195
61 (3) 9580 9288

www.ameda.com

Distribution in Europe:
To contact your local distributor for product information or feedback, visit www.ameda.com/where/international.aspx

CE 0086 (applicable to Product Code 171481P, 173581)

EMERGO EUROPE
Molenstraat 15
2513 BH, The Hague
The Netherlands
Phone: +31.70.345.8570
Fax: +31.70.346.7299

Ameda, HygieniKit and associated logos are trademarks of Evenflo Company, Inc.
©2009 Evenflo Company, Inc.
25700448-1209

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