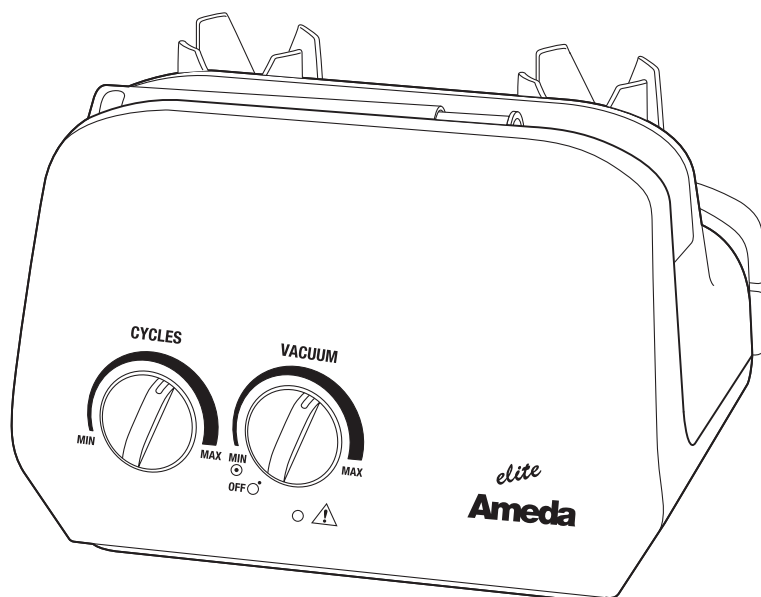
(EN) Ameda® Elite™ Breast Pump Instructions	2
(DE) Gebrauchsanleitung Ameda Elite Milchpumpe	11
(FR) Mode d'emploi du Tire-lait Ameda Elite	17
(IT) Istruzioni per il tiralatte Ameda Elite	23
(ES) Instrucciones del sacaleches Ameda Elite	29
(PL) Instrukcja obsługi laktatora Ameda Elite	35
(PT) Instruções da Bomba de Peito Ameda Elite	41
(NL) Gebruiksaanwijzingen voor de Ameda Elite -borstpomp	47
(DK) Vejledning til brystpumper Ameda Elite	53
(NO) Anvisninger for Ameda Elite brystpumpe	59
(SE) Instruktioner till Ameda Elite bröstpump	65
(FI) Ameda Elite -rintapumpun käyttöohjeet	71
(GR) Οδηγίες για το θήλαστρο Ameda Elite	77
(TR) Ameda Elite Göğüs Pompası için Talimatlar	83
(RO) Instrucțiuni pentru pompa de sân Ameda Elite	89

CONTENTS

Important Safeguards	2	Cleaning	7
Intended Use	2	Ameda Elite Breast Pump	7
Ameda Elite Breast Pump Diagram	3	For At-Home/End-Users	7
Ameda Elite Breast Pump Trolley	5	For Hospitals/Rental Stations	7
Set Up and Assembly	6	Troubleshooting	7
Ameda Pumpset Milk Collection System	6	Maintenance	7
Before Every Use	6	Technical Information	7
Single and Dual Pumping	6	Product Disposal	10
Operating the Elite Breast Pump	6	Warranty Information	10
Setting Suction	6		
Setting Speed	6		
Turning Pump OFF	6		
Removing Ameda Pumpset from Breast(s)	6		

IMPORTANT SAFEGUARDS

When using electrical products, especially if children are present, the following basic safety precautions should always be maintained.

READ ALL INSTRUCTIONS BEFORE USING

Danger: To reduce the risk of electrocution:

- Always unplug electrical devices immediately after use.
- Do not use while bathing, showering or swimming.
- Do not place or store where product can fall or be pulled into bathtub, sink or pool.
- Do not place or drop product into water or other liquid.
- Do not reach for electrical product that has fallen into water. Unplug from wall outlet immediately.

Warning: To reduce the risk of burns, electrocution, fire, or injury to persons:

- Never leave product unattended when plugged into electrical outlet.
- Close supervision is necessary when product is used near children or invalids.
- Use product only for intended use as described in this manual.
- Do not use attachments or other milk collection kits other than those recommended by manufacturer.
- Never operate if product has damaged cord or plug, is not working properly, or has been dropped, damaged, or becomes wet.
- Keep cord and all attachments away from heated surfaces.
- Never use while sleeping or drowsy.
- Do not use outdoors with cord.
- Do not operate where aerosol spray products are being used or oxygen is being administered.

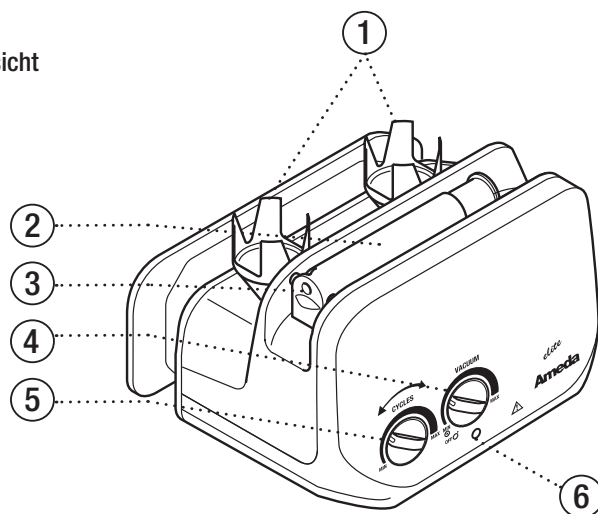
Caution: To reduce the risk of fire or chemical burn:

- Do not disassemble or heat above 100°C (212°F).

Intended Use

The Ameda Elite Breast Pump is intended to facilitate the expression and collection of breast milk from a lactating woman for the purpose of feeding the collected milk to a child.


- (EN) Pump Diagrams – Front and Top View
 (DE) Schemazeichnung der Pumpe – Vorderansicht und Draufsicht
 (FR) Schémas du tire-lait : – Vue de face et de dessus
 (IT) Schema del tiralatte – Vista frontale e superiore
 (ES) Diagramas del sacaleches – Vista frontal y superior
 (PL) Schematy laktatora – Widok z przodu i z góry
 (PT) Diagramas da Bomba – Vista de Frente e de Cima
 (NL) Pumpschema's – Voor- en bovenaanzicht
 (DK) Pumpediagrammer – Set forfra og ovenfra
 (NO) Pumpediagrammer – Sett for- og ovenfra
 (SE) Pumpdiagram – Sedd fram- och ovanifrån
 (FI) Pumpun kaaviokuvat – Näkymä edestä ja päältä
 (GR) Διαγράμματα θηλάστρου – Μπροσινή και πάνω πλευρά
 (TR) Pompa Şemaları – Önden ve Üstten Görünüm
 (RO) Structura pompei – Vedere frontală şi de sus

- | | | |
|--|--|--|
| <p>1. Milk Bottle/Breast Flange Holders
 Halterung für Milchflasche/Brustglocke
 Support du bouteille/du manchon de sein
 Bottiglia per il latte/supporto della flangia per il seno
 Sujetador de frasco de leche/copa
 Uchwyt na butelki/lejki
 Frasco de Leite/Suporte da Flange Mamária
 Melkfles/Kolfstukhouder
 Holder til mælkeflaske/brystflange
 Melkeflaske/Brystkrageholder
 Hållare för mjölkflaska/bröstkupa
 Maitopullon/rintalaipan pidike
 Βάση φιάλης γάλακτος/χοάνης
 Süt Şişesi/Göğüs Flanşı Tutucusu
 Colector lapte/suport cupă colectoare pentru sân</p> | <p>2. Piston/Cylinder
 Kolben/Zylinder
 Ensemble piston/cylindre
 Pistone/Cilindro
 Émbolo/Cilindro
 Tłok/cylinder
 Pistão/Cilindro
 Zuiger/Cilinder
 Stempel/cylinder
 Stempel/sylinder
 Kolv/cylinder
 Mäntä/sylinteri
 Έμβολο/Κύλινδρος
 Piston/Silindir
 Piston/Cilindru</p> | <p>3. Tubing Adapter Port
 Schlauchadapteranschluss
 Port de l'adaptateur de tube
 Alloggiamento per l'adattatore del tubo
 Puerto del adaptador del tubo
 Wejście złączki przewodów
 Porta do Adaptador do Tubo
 Poort verbindingsstuk voor slang
 Slangedapterport
 Slangadapterport
 Slangadapteranslutning
 Putkisovittimen liitäntäaukko (PP)
 Θύρα προσαρμογέα σωλήνα
 Hortum Adaptörü Portu
 Ataşarea portului adaptorului</p> |
| <p>4. 'VACUUM' Dial
 Einstellregler „VAKUUM“
 Molette « VIDE »
 Selettore VUOTO
 Indicador “SUCCIÓN”
 Pokrętło „VACUUM” (Podciśnienie)
 Botão VÁCUO
 'VACUUM'-knop
 “VAKUUM”-indstillingsknap
 VAKUUM-skive
 VACUUM-ratt
 ”ALIPAINEN” säädin
 Διακόπτης KENOY
 'VAKUM' Kadrani
 Variatorul „VACUUM”</p> | <p>5. 'CYCLES' Dial
 Einstellregler „ZYKLEN“
 Molette « CYCLES »
 Selettore CICLI
 Indicador “CICLOS”
 Pokrętło „CYCLES” (Cykle)
 Frasco CICLOS
 'CYCLES'-knop
 ”CYKLUS”-indstillingsknap
 SYKLUS-skive
 CYCLES-ratt
 ”JAKSON” säädin
 Διακόπτης KYKΛΩΝ
 'DÖNGÜ' Kadrani
 Variatorul „CYCLES”</p> | <p>6. Power Light
 Netzanzeige
 Témoin lumineux de mise sous tension
 Spia di accensione
 Luz de encendido
 Kontrolka zasilania
 Luz de Ligado
 Lampje stroomtoevoer
 Strømlampe
 Strømlampe
 Strömbrytarlampa
 Virran merkkivalo
 Φωτεινή ένδειξη λειτουργίας
 Güç Işığı
 LED de alimentare</p> |

- Bottom View**
 Ansicht von unten
 Vue de dessous
 Vista inferiore
 Vista inferior
 Widok z dołu
 Vista Inferior
 Onderaanzicht
 Set nedefra
 Sett nedenfra
 Sedd nerifrån
 Näkymä alhaalta
 Κάτω πλευρά
 Altan Görünüm
 Vedere de jos
- 7. Pump Feet**
 Pumpenfüße
 Pieds du tire-lait
 Piedini del tiralatte
 Patas del sacaleches
 Nóżki laktatora
 Apoios da Bomba
 Voeten van het toestel
 Pumpefødder
 Pumpeføtter
 Pumpfot
 Pumpun tukijalakset
 Πέλματα αντήλας
 Pompa Ayağı
 Baza pompei


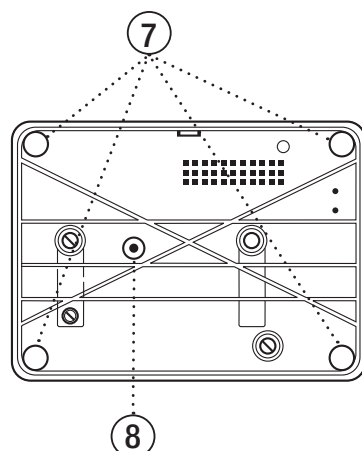
- 8. Trolley Mounting Hole**
 Loch für Montage am Wagen
 Orifice de fixation sur le chariot
 Foro per l'inserimento del carrello
 Orificio de montaje del carrito
 Otwór mocowania do stojaka
 Orifício de Montagem do Carrinho
 Montagegaatje trolley
 Monteringsåbning til vogn
 Vognmonteringshull
 Monteringshål för vagn
 Työntökärryn kiinnitysreikä
 Οπή σύνδεσης κυλιόμενης βάσης
 Taşıma Düzeneği Montaj Deliği
 Fanta de montare a suportului

Rear View
Ansicht Rückseite
Vue de derrière
Vista posteriore
Vista trasera
Widok z tyłu
Vista Posterior
Achteraanzicht
Set bagfra
Sett bakfrå
Sedd bakifrån
Näkymä takaa
Κάτω πλευρά
Arkadan Görünüm
Vedere din spate

Elite with Detachable Power Cord
Elite mit abnehmbarem Stromkabel
Elite avec cordon d'alimentation amovible
Elite con cavo di alimentazione removibile
Extractor Elite con cable eléctrico desmontable
Laktator Elite z odłączanym przewodem zasilającym
Elite Com Cabo Eléctrico Amovível
Elite met afneembaar snoer
Elite med aftagelig el-ledning
Elite med avtakbar strømlledning
Elite med löstagbar strömkabel
Elite ja irrotettava virtajohto
Elite με αφαιρούμενο καλώδιο ισχύος
Çıkarılabilir Güç Kablolu Elite
Elite cu cablu de alimentare detaşabil

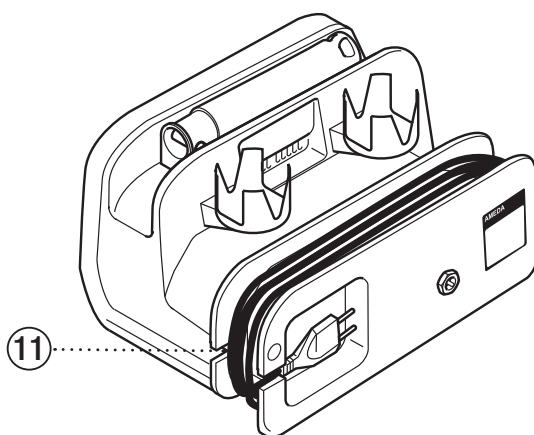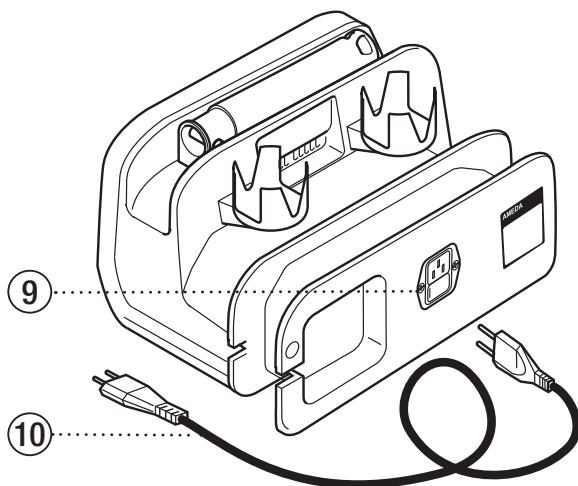Elite with Built-in Power Cord
Elite mit integriertem Stromkabel
Elite avec cordon d'alimentation intégré
Elite con cavo di alimentazione incorporato
Extractor Elite con cable eléctrico incorporado
Laktator Elite z wbudowanym przewodem zasilającym
Elite com Cabo Eléctrico Integrado
Elite met ingebouwd snoer
Elite med indbygget el-ledning
Elite med innebygget strømlledning
Elite med inbyggd strömkabel
Sisäisellä virtajohtolla varustettu Elite
Elite με ενσωματωμένο καλώδιο ισχύος
Sabit Güç Kablolu Elite
Elite cu cablu de alimentare încorporat


- 9. AC Power Inlet**
Netzstromsteckplatz
Prise d'alimentation CA
Ingresso dell'alimentazione
Entrada de CA
Gniazdo zasilania sieciowego
Entrada de Corrente AC
Wisselstroömingang
Vekselstrømskontakt
Vekselstrøminngang
Strömanslutning
AC-virrantuloaukko
Είσοδος ισχύος AC
AC Güç Girişi
Intrare curent alternativ

- 10. Detachable Power Cord**
Abnehmbares Stromkabel
Cordon d'alimentation amovible
Cavo di alimentazione removibile
Cable eléctrico desmontable
Odłączany przewód zasilający
Cabo Eléctrico Amovível
Afneembaar snoer
Aftagelig el-ledning
Avtakbar strømlledning
Löstagbar strömkabel
Irrotettava virtajohto
Αφαιρούμενο καλώδιο ισχύος
Çıkarılabilir Güç Kablosu
Cablu de alimentare detaşabil

- 11. Power Cord**
Stromkabel
Cordon d'alimentation
Cavo di alimentazione
Cable eléctrico
Przewód zasilający
Cabo Eléctrico
Snoer
El-ledning
Strømlledning
Strömkabel
Virtajohto
Καλώδιο ισχύος
Güç Kablosu
Cablu de alimentare

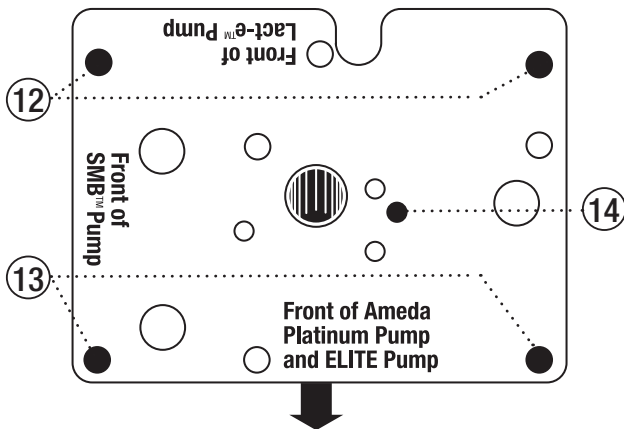- (EN) **Trolley Plate – Top View** (*optional accessory*)
 (DE) **Wagenplatte – Aufsicht** (*optionales Zubehör.*)
 (FR) **Plaque du chariot : – vue de dessus** (*accessoire facultatif*)
 (IT) **Piastra del carrello – Vista superiore** (*accessorio opzionale*)
 (ES) **Placa del carrito - Vista superior** (*accesorio opcional*)
 (PL) **Płyta stojaka – widok z góry** (*wyposażenie dodatkowe*)
 (PT) **Base do Carrinho – Vista Superior** (*acessório opcional*)
 (NL) **Trolleyplaat – bovenaanzicht** (*optioneel accessoire*)
 (DK) **Vognplade – set oppefra** (*ekstraudstyr*)
 (NO) **Vognplate – Sett ovenfra** (*frivillig tilbehør*)
 (SE) **Vagnplatta – sedd ovanifrån** (*tillbehör som tillval*)
 (FI) **Työntökärryn alusta – Näkymä ylhäältä** (*valinnainen lisäosa*)
 (GR) **Έλασμα κυλιόμενης βάσης – Πάνω πλευρά** (*προαιρετικό αξεσουάρ*)
 (TR) **Taşıma Düzeneği Plakası – Üstten Görünüm** (*isteğe bağlı aksesuar*)
 (RO) **Placa suportului – Vedere de sus** (*accesoriu opțional*)

Front of Ameda Platinum® Pump and Elite Pump
 Vorderseite der Ameda Platinum-Pumpe und der Elite-Pumpe
 Façade des pompes Ameda Platinum et Elite
 Parte frontale della pompa Ameda Platinum e della pompa Elite
 Frontal del extractor de leche Ameda Platinum y del extractor de leche Elite
 Przednia część laktatora Ameda Platinum i Elite
 Frente das Bombas Ameda Platinum e Elite”
 Frontje van de Ameda Platinum en Elite borstkolf
 Forsiden af Ameda Platinum-pumpen og Elite-pumpen
 Ameda Platinum-pumpe og Elite-pumpe sett forfra
 Ameda Platinum-pumpens och Elite-pumpens framsida
 Ameda Platinum -pumppu ja Elite-pumppu edestä
 Μπροστινή πλευρά θηλάστρου Ameda Platinum και θηλάστρου Elite
 Ameda Platinum Pompanın ve Elite Pompanın Ön Yüzü
 Partea din față a pompei Ameda Platinum și a pompei Elite

12. Pump Rear Foot Location
 Anbringungsstelle hinterer Pumpenfuß
 Emplacement du pied arrière du tire-lait
 Posizione dei piedini posteriori del tiralatte
 Ubicación de la pata trasera del sacaleches
 Lokalizacja tylnych nóżki laktatora
 Localização do Apoio Posterior da Bomba
 Plaats voor achterste voet
 Sted for pumpens bagfod
 Plassering på pumpens bakfot
 Placering av pumpens bakre fot
 Pumpun takajalaksen sijainti
 Θέση πίσω πέλματος θηλάστρου
 Pompa Arka Ayak Yeri
 Amplasarea suportului din spate a pompei
13. Pump Front Foot Location
 Anbringungsstelle vorderer Pumpenfuß
 Emplacement du pied avant du tire-lait
 Posizione dei piedini anteriori del tiralatte
 Ubicación de la pata frontal del sacaleches
 Lokalizacja przedniej nóżki laktatora
 Localização do Apoio Frontal da Bomba
 Plaats voor voorste voet
 Sted for pumpens forfod
 Plassering på pumpens fremfot
 Placering av pumpens främre fot
 Pumpun etujalaksen sijainti
 Θέση μπροστινού πέλματος θηλάστρου
 Pompa Ön Ayak Yeri
 Amplasarea suportului din față a pompei
14. Pump Mounting Hole
 Loch für Montage der Pumpe
 Orifice de fixation du tire-lait
 Foro per l'inserimento del tiralatte
 Orificio de montaje del sacaleches
 Otwór do mocowania laktatora
 Orifício de Montagem da Bomba
 Montagegatje pomp
 Monteringsåbning for pumpen
 Pumpemonteringshull
 Monteringshål för pump
 Pumpun kiinnitysreikä
 Οπή σύνδεσης θηλάστρου
 Pompa Montaj Deliği
 Orificiu de montare a pompei

(EN) Attaching the Ameda Elite Breast Pump to Trolley (if applicable)

Trolley

Trolley is available as an accessory item (Stock No. 17811).

- Attach pump to trolley prior to plugging cord into wall outlet.
- Align front of the Ameda Elite Breast Pump with wording, “Front of Ameda Platinum Pump and Elite Pump” found on top of trolley mounting plate.
- Lower pump onto plate, inserting pump’s four feet into holes, one in each corner of trolley mounting plate.
- Locate mounting hole with wording, “Ameda Platinum Pump and Elite Pump Mounting” found on underside of trolley mounting plate.
- Use mounting screw for the Ameda Elite Breast Pump supplied with trolley to secure pump to trolley base.

Set Up and Assembly

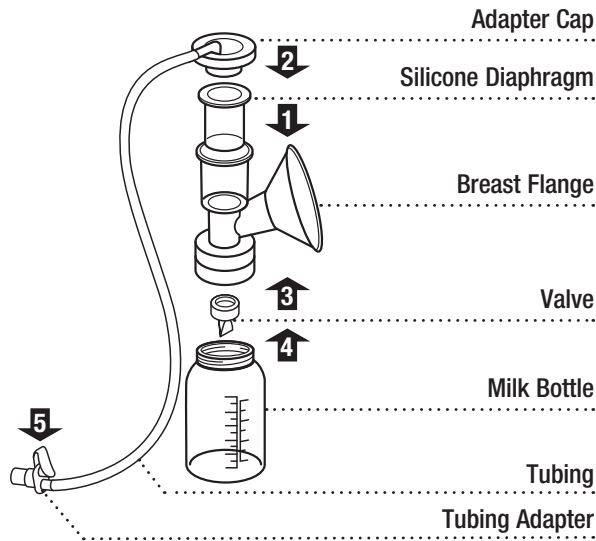
Ameda Pumpset Milk Collection System

Before Every Use


Always wash hands well with soap and water before handling the Ameda Elite Breast Pump and Ameda Pumpset Milk Collection System.

IMPORTANT NOTE: ONLY USE THE AMEDA PUMPSET MILK COLLECTION SYSTEM ON AMEDA ELECTRIC BREAST PUMPS.


Prior to starting pump, assemble and connect the Ameda Pumpset Milk Collection System, per included instructions. If the Ameda Pumpset Milk Collection System is sterile, use as is. If not sterile, see the Ameda Pumpset Milk Collection System cleaning instructions.

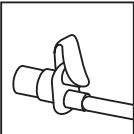
The Ameda Pumpset Milk Collection System can be used to single or dual pump.

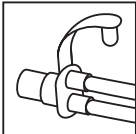To single pump:
Remove one tube and close tubing adapter.

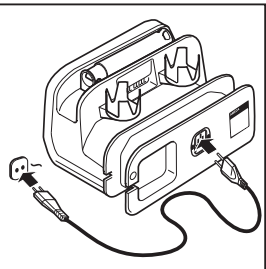
To dual pump:
Both tubes should be attached to tubing adapter.


Elite with Detachable Power Cord

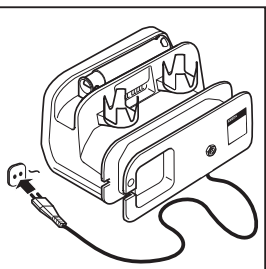- Attach Ameda supplied power cord to AC power inlet on rear of pump.
- Connect power cord to wall outlet.


Elite with Built-in Power Cord

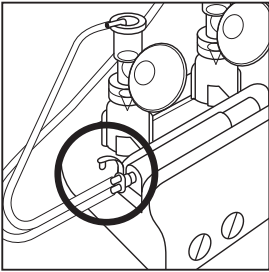- Connect power cord to wall outlet.


- Insert tubing adapter into tubing adapter port on piston/cylinder.

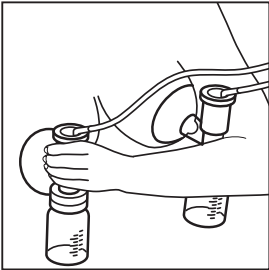Operating the Pump

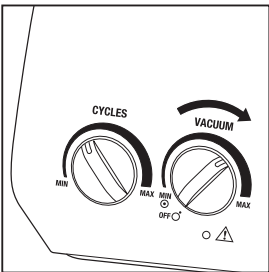
- Center nipple(s) in breast flange(s) and fill breast flange(s) with the breast(s) to create an air seal.
- Turn the pump ON by turning the 'VACUUM' dial to the right.

Setting Suction

- Increase the suction by turning the 'VACUUM' dial to the right. Increase suction until reaching maximum comfort level.
- If discomfort is felt in nipple(s) or breast(s), decrease suction. If discomfort is felt at all suction settings, turn pump

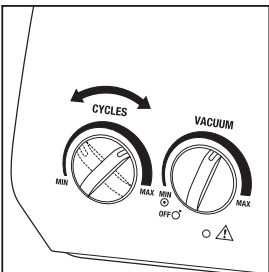
off, insert a finger between breast(s) and breast flange(s) to break suction, remove breast flange(s) from breast(s). Discontinue pumping and contact a healthcare provider.

Setting Speed

- Start with the 'CYCLES' dial all the way to the right to increase the speed and trigger a milk ejection reflex (let-down).
- When milk flows, (let-down) turn the 'CYCLES' dial to the left to slow down the speed to help drain the breast

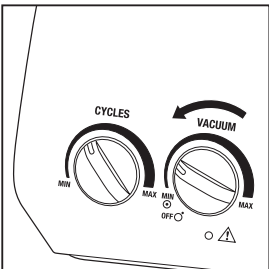faster. When milk flow slows, turn the 'CYCLES' dial all the way to the right to help start another (let-down). Repeat as milk flow changes to pump more milk.


Turning Pump OFF

- When finished pumping, turn the 'VACUUM' dial all the way to the left to turn the pump off.


Removing Ameda Pumpset from Breast(s)

- After pump is switched off, if necessary, insert a finger between breast(s) and breast flange(s) to break suction.
- Remove breast flange(s) from breast(s).

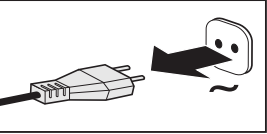
Cleaning the Ameda Elite Breast Pump

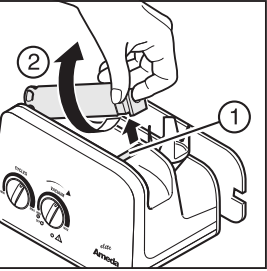- Disconnect the pump power cord from wall outlet.

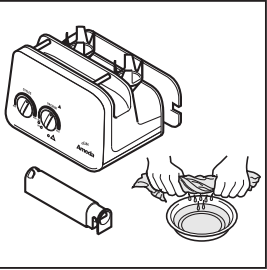
- Remove piston cylinder by placing index finger into the tube, lifting upwards and turning right to release from attachment bars.

For At-Home/End-Users
After each pumping, wipe pump with cloth dampened with warm, soapy water.

For Hospitals/Rental Stations
Clean between users. Spray with recommended disinfectant, such as CaviCide™. Wipe with soft cloth. Do not use products containing phenol.

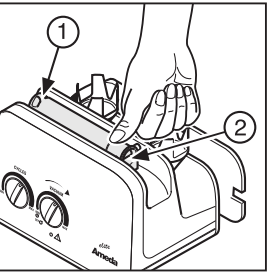Do not use bleach on pump.

- Place piston cylinder back onto attachment bars and press gently into place with thumb.

Troubleshooting

Pump Does Not Turn ON

- Ensure proper power cord connection to pump.
- Ensure power cord connection to wall outlet.
- Ensure outlet has power.

Low or No Suction

- Check breast flange for adequate seal against breast.
- Check that the Ameda Pumpset Milk Collection System is properly assembled.
- Check that adapter cap is firmly snapped onto top of breast flange.
- Ensure silicone diaphragm is in place inside top of breast flange.
- Ensure tubing adapter is properly fitted into tubing adapter port on pump.
- Inspect valve for stretching or tears. If seen, replace valve.
- Ensure valve is pushed firmly into place in lower portion of breast flange.
- Ensure one side of tubing adapter is closed when single pumping.

Maintenance

It is recommended the Ameda Elite Breast Pump be inspected at least once every year to include the following:

- Visually check pump housing for cracks, breakage, or damage and ensure pump feet are present.
- Visually check for power cord damage.
- Check for proper operation of controls and buttons.
- Check there are no missing screws.
- Check tubing adapter port on piston/cylinder is free from obstruction.

Quick Functional Check

The following checks/tests should be performed prior to use by a new user:

- Visual checks for power cord damage.
- Check for abnormal noise.
- Check for proper operation of controls.

Replacing Detachable Power Cord (if applicable)

- Power cord is detachable from power inlet.
- If power cord condition is in doubt, only replace with Ameda supplied 2 conductor cord.

Technical Information

Main voltage

Elite with Detachable Power Cord 100VAC to 240VAC

Elite with Built-In Power Cord 100/120V ~ or 220/240V ~ /AC

Frequency 50 / 60 Hz

Power consumption 30 VA

Main fuse T 1.6A

Protection class Class II

Double isolated

Safety level

Drip protection IP 21

Operational conditions +10 - +40°C
10 - 90% rel. humidity

Transport and - 40 - +70°C

Storage conditions 10 -100% rel. humidity

Weight 6.4 lbs/2.9 kg

Dimensions 10.0"L x 9.0"W x 7.5"H
22.2cmL x 26.7cmW x 18.4cmH

Vacuum max. [mbar] [mm Hg]

Single pumpset 350 (-10%) 262 (-10%)

Double pumpset 310 (-15%) 232 (-15%)

Variable cycle 30 - 60 / Min

Conforming to

CE 0086

E325189 MEDICAL EQUIPMENT
WITH RESPECT TO ELECTRIC SHOCK,
FIRE AND MECHANICAL HAZARDS ONLY.
IN ACCORDANCE WITH
IEC60601-1: 1988 + A1:1991 + A2:1995,
IEC60601-1-2, CAN/CSA C22.2 No. 601.1-M90

Evenflo Company, Inc.
1801 Commerce Drive
Piqua, Ohio 45356
USA

EMERGO EUROPE
Molenstraat 15
2513 BH, The Hague
The Netherlands
Phone: +31.70.345.8570
Fax: +31.70.346.7299

MEDICAL ELECTRICAL EQUIPMENT needs special precautions regarding EMC and needs to be installed and put into service according to the EMC provided in these Instructions for use.

Portable and mobile RF communications equipment can affect MEDICAL ELECTRICAL EQUIPMENT.

Warning: The use of ACCESSORIES, transducers and cables other than those specified, with the exception of transducers and cabled sold by the MANUFACTURER of the Ameda Elite

Breast Pump as replacement parts for internal components, may result in increased EMISSIONS or decreased IMMUNITY of the Ameda Elite Breast Pump.

Warning: The Ameda Elite Breast Pump should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, the Ameda Elite Breast Pump should be observed to verify normal operation in the configuration in which it will be used.

Guidance and manufacturer's declaration – electromagnetic emissions		
The Ameda Elite Breast Pump is intended for use in the electromagnetic environment specified below.		The customer or the user of the Ameda Elite Breast Pump should assure that it is used in such an environment.
Emissions test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The Ameda Elite Breast Pump uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment. The Ameda Elite Breast Pump is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
RF emissions CISPR 11	Class B	
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/flicker emissions IEC 61000-3-3	Complies	

Guidance and manufacturer's declaration – electromagnetic immunity			
The Ameda Elite Breast Pump is intended for use in the electromagnetic environment specified below.		The customer or the user of the Ameda Elite Breast Pump should assure that it is used in such an environment.	
IMMUNITY test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 6 kV contact ± 8 kV air	± 6 kV contact ± 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	± 2 kV for power supply lines/± 1 kV for input/output lines	± 2 kV for power supply lines/± 1 kV for input/output lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV line(s) to line(s) ± 2 kV line(s) to earth	± 1 kV line(s) to line(s) ± 2 kV line(s) to earth	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5% U _T (>95% dip in U _T) for 0,5 cycle 40% U _T (60% dip in U _T) for 5 cycles 70% U _T (30% dip in U _T) for 25 cycles <5% U _T (>95% dip in U _T) for 5 s	<5% U _T (>95% dip in U _T) for 0,5 cycle 40% U _T (60% dip in U _T) for 5 cycles 70% U _T (30% dip in U _T) for 25 cycles <5% U _T (>95% dip in U _T) for 5 s	Mains power quality should be that of a typical commercial or hospital environment. If the user of the Ameda Elite Breast Pump requires continued operation during power mains interruptions, it is recommended that the Ameda Elite Breast Pump be powered from an uninterruptible power supply.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
NOTE: U _T is the a.c. mains voltage prior to application of the test level.			

Guidance and manufacturer's declaration – electromagnetic immunity			
The Ameda Elite Breast Pump is intended for use in the electromagnetic environment specified below.		The customer or the user of the Ameda Elite Breast Pump should assure that it is used in such an environment.	
IMMUNITY test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Conducted RF IEC 61000-4-6 Radiated RF IEC 61000-4-3	3 Vrms 150 kHz to 80 MHz 3 V/m 80 MHz to 2,5 GHz	3 Vrms 3 V/m	Portable and mobile RF communications equipment should be used no closer to any part of the Ameda Elite Breast Pump, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance $d = 1,2\sqrt{P}$ $d = 1,2\sqrt{P}$ 80 MHz to 800 MHz $d = 2,3\sqrt{P}$ 800 MHz to 2,5 GHz Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey,^a should be less than the compliance level in each frequency range.^b Interference may occur in the vicinity of equipment marked with the following symbol: 
NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.		survey should be considered. If the measured field strength in the location in which the Ameda Elite Breast Pump is used exceeds the applicable RF compliance level above, the Ameda Elite Breast Pump should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the Ameda Elite Breast Pump. ^b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.	
NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			
^a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site			

Recommended separation distances between portable and mobile RF communications equipment and the Ameda Elite Breast Pump			
The Ameda Elite Breast Pump is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Ameda Elite Breast Pump can help prevent electromagnetic interference by maintaining a minimum		distance between portable and mobile RF communications equipment (transmitters) and the Ameda Elite Breast Pump as recommended below, according to the maximum output power of the communications equipment.	
Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d = 1,2\sqrt{P}$	80 MHz to 800 MHz $d = 1,2\sqrt{P}$	800 MHz to 2,5 GHz $d = 2,3\sqrt{P}$
0,01	0,12	0,12	0,23
0,1	0,38	0,38	0,73
1	1,2	1,2	2,3
10	3,8	3,8	7,3
100	12	12	23
For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.		NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies. NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.	

Product Disposal

This product contains electrical and electronic equipment.

1. As with other electrical equipment, the internal components of this product may contain hazardous materials. To avoid potential contamination of the environment, it is important that you dispose of this product according to local or regional waste administration systems and regulations.
2. Do not dispose of as unsorted municipal waste.
3. The crossed-out wheeled bin symbol shown indicates separate collection for electrical and electronic equipment.
4. You may contact us for further information regarding the environmental performance of this product.

Limited Warranty: First User, Original Manufactured Product Only

DOES NOT APPLY TO REFURBISHED OR RECONDITIONED PRODUCTS.

The Ameda Elite Electric Breast Pump ("Product") is warranted to the original user--whether institutional or individual ("First User")--only.

From the date of First User's purchase of this Product, Evenflo Company, Inc. ("Evenflo") warrants the Product to the First User against defects in material or workmanship for two years on the pump mechanism. Evenflo's sole obligation under this limited warranty shall be to repair or replace, at Evenflo's option, any Product that is determined to be defective by Evenflo and determined to be covered by this express limited warranty. Repair or replacement under this limited warranty is the sole and exclusive remedy of the First User. Proof of purchase in the form of a receipted invoice or bill of sale evidencing that the Product is within the warranty period must be presented to obtain warranty service. This limited warranty is extended by Evenflo ONLY to the First User and is not assignable or transferable. For warranty service contact Ameda at 1.866.99.AMEDA(26332).

EXCEPT TO THE EXTENT PROHIBITED BY APPLICABLE LAW, ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE ON THE PRODUCT IS HEREBY DISCLAIMED. EVENFLO SHALL NOT BE LIABLE FOR ANY INCIDENTAL, CONSEQUENTIAL, SPECIAL OR PUNITIVE DAMAGES OF ANY KIND FOR BREACH OF THE EXPRESS LIMITED WARRANTY ON THE PRODUCT OR ANY WARRANTY IMPLIED BY OPERATION OF LAW, OTHER THAN THE EXPRESS LIMITED WARRANTY SET FORTH ABOVE, THERE ARE NO OTHER WARRANTIES THAT ACCOMPANY THIS PRODUCT AND ANY ORAL, WRITTEN OR ANY OTHER EXPRESS REPRESENTATION OF ANY KIND IS HEREBY DISCLAIMED.

Non-Applicability of Warranty

The above warranty shall not apply to consumable materials and attachment to the Product. These include:

- (i) the power cord; and
- (ii) the Ameda Pumpset Milk Collection System (sold separately).

In addition to any other limitations on the warranty discussed above, the warranty shall expire and be of no force or effect upon the occurrence of any of the following:

- (i) the cause of damage to the Product is due to improper handling or use
- (ii) the Product is not properly maintained by the First User, as determined by Evenflo
- (iii) non-Ameda approved accessories or spare parts are used with the Product, as determined by Evenflo
- (iv) the Product is repaired by a non-approved Evenflo service provider.

If any of the above occurs, Evenflo shall not be responsible for damage to the Product, parts thereof or injuries arising therefrom, either directly or indirectly. The warranty set forth above replaces all prior warranties with respect to the purchased Product, whether in writing or otherwise.

Within the United States: Call Ameda Customer Service at 1.866.99.AMEDA(26332) for any questions regarding this limited warranty and its applicability to your product.

Outside the United States: Call your local distributor or location where you purchased the Product. For a listing of distributors in your country, please visit www.ameda.com.

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©2012 Evenflo Company, Inc.
Ameda, Ameda Platinum, Elite, SMB, Lact-e, and associated logos are trademarks of Evenflo Company, Inc.
CaviCide is a registered trademark of Metrex Research Corporation.